# Erkoton 20/30

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## EN Working instruction

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
ERKOTON 20/30, mouldable, addition cured silicone material, highly elastic, tear resistant, easy to grind, two Shore A hardnesses available, comfortable to wear.

### Application:

For the construction of pressure reliefs, orthoses, interdigital toe wedges or toe compensation. One level measuring spoon (transparent) of hardener mass is mixed with one level spoon (orange) of ERKOTON 20/30. Mix quickly and thoroughly within 30 sec. until a uniform colour is achieved. Mixing the material for a longer time will lead to air bubbles and folds which will reduce the durability of the orthosis considerably. After mixing, the material should be formed to a ball or strip and moulded at the foot. Modelling should be finished within 2 min. to avoid - as already mentioned above - air bubbles and folds. The patient should be asked to stand during the forming of the orthosis. ERKOTON can be ground by abrasive belts, pumicers or multiple-discs. To polish the surface we recommend to use LSKO polishing discs fine, white (art. no. 413 678).

### Dosage:

To 1 level measuring spoon of ERKOTON 20/30, 1 level measuring spoon of hardener mass is added. Proportions of mixture 1 : 1.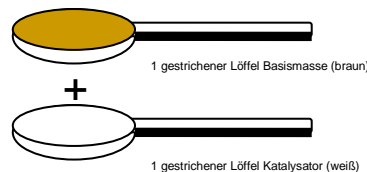

### Technical Data:

- mixing time 30 - 45 sec.
- total working time 2 - 2.5 min.
- setting time 6 - 8 min.
- app. 20 or 30 Shore A
- Type 0 - very high consistency - mouldable (EN 24823)
- store ERKOTON 20/30 in a dry place at room temperature
- disinfection of the orthosis is possible
- washable
- Durability - see imprint on package.

### Important:

Close container immediately after use! Use coloured spoon and marked lid only with marked container! Do not bring in contact with latex gloves, disinfectant, foot cream or something like that.

### Availability:

- 413 821 ERKOTON 20, 56 ml, soft
- 413 831 ERKOTON 30, 56 ml, medium
- 413 820 ERKOTON 20, 200 ml, soft
- 413 830 ERKOTON 30, 200 ml, medium
- 413 825 ERKOTON 20, 800 ml, soft
- 413 835 ERKOTON 30, 800 ml, medium

\* corresponds to the guide 93/42/EEC for medical devices