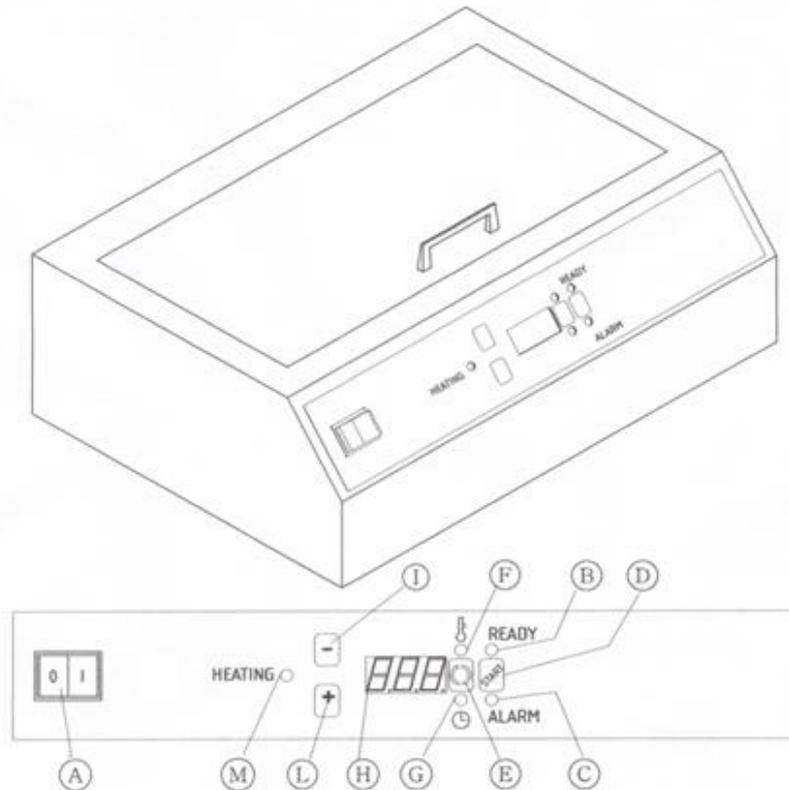
**HEISSLUFT
Sterilisatoren**

**DRY
Sterilizer**

en - USER MANUALpage 2

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DRY STERILIZER- USER MANUAL

REMARKS:

WARNING: Burn hazard: The inner parts of the chamber and the door reach high temperatures

- ◆ These operator's instructions have to be carefully kept near the equipment, in order to be consulted by all users.
- ◆ Before using the machine for the first time we suggest to perform a sterilization cycle with no load
- ◆ The use of this device is limited to qualified staff
- ◆ Before the start, please check that your load withstands without any damages at the set temperature (+10% for safety).
- ◆ The equipment is to be used dry, do not introduce water or other liquids.
- ◆ In case of black-out the sterilizing cycle is not valid
- ◆ In case of lack of voltage or lighting of the alarm pilot lamp the equipment stops the sterilization cycle.
- ◆ We recommend not to lean any object on the equipment

- ◆ The material should be placed in such a way that the hot air can flow freely around the material that is to be sterilised.
- ◆ We recommend to clean and dry carefully the material to sterilize.
- ◆ Temperature oscillations of few °C, during the cycle, have to be considered normal.

- ◆ In case more consecutive cycles the apparatus could achieve elevated external temperatures.
- ◆ **It is suggested to periodically check the proper operation of the appliance with the appropriate indicator (chemical and/or biological for dry sterilization) to ensure correct sterilization.**

The equipment is only designed for the sterilisation of metal objects (scissors, stitch-removers, nail clippers etc), glass or other material capable of withstanding the temperature of the machine.

- ◆ It is inadvisable to place gauze, textiles and packaging inside for sterilisation.

Description of the equipment :

Referring to the figure of page 2 the sterilizer components are described here below:

- (A) **Master switch**
- (B) **"Ready machine" yellow pilot lamp**
This lamp is on when the sterilizer is ready and is waiting the start of a cycle
- (C) **Alarm red lamp** 
This pilot lamp turns on in order to signal some possible anomalies and it is matched to a BIP sound.
To set aside the alarm push the START button (see Alarms Table).
- (D) **START push button**
The function of this push button is to start a cycle, if the pilot lamp (B) is turned on.
The **START** push button works also as RESET key, if it is pressed during a cycle it is cancelled. Moreover, if an alarm has intervened, by pressing the **START** push button, the alarm is cancelled
- (E) **Switch push button** 
The function of this push button is to alternate, on the display (H), the visualization of temperature and sterilization time.
- (F) **Temperature pilot lamp** 
When this pilot lamp is turned on it shows that the value read and/or set on the display (H) is referred to the temperature.
Attention: at temperatures below 140°C the display may show different temperatures from the real one.
- (G) **time pilot lamp** 
When this pilot lamp is turned on it shows that the value read and/or set on the display (H) is referred to the sterilization length of time, heating excluded.
If it flashes it means that INFINITE time has been set

- **(H) display**

Cycle parameters are visualized on the display : temperature inside the chamber, sterilization time, set temperature.

In order to reset the cycle data, this must be stopped.

- The temperature can be set between 140°C and 200°C,
- Time can be set between 1 minute and 600 minutes or infinite time.

To set the INFINITE time the push button **+** must be kept pressed up to 600 and then it is necessary to wait few seconds.

When the writing "INF" appear on the display wait 3 seconds for automatic storage.

When INFINITE time is selected the Lamp Time (G) flashes.

To remove INFINITE time press the button **—**.

ATTENTION: AT INFINITE TIME THE EQUIPMENT CAN BE SWITCHED OFF ONLY MANUALLY.

- **(I) (L) keys**  

The function of these keys is to modify the temperature setting and the sterilization time, increasing or decreasing the value. By pressing the **+** key once, with the sterilizer waiting (B pilot lamp turned on) the set temperature value or the set sterilization time is visualized on the display.

By pressing the **+** or **—** keys it is possible to modify the value.

After reaching the desired value, wait 3 seconds for automatic acquisition.

If one of these keys is pressed during the cycle, the set values of time or temperature can be read on the display but it is not possible to modify them.

- **(M) Heating pilot lamp**

This lamp indicates the switching on of the heating elements of the equipment

USER'S INSTRUCTIONS

Before setting sterilization time and temperature it is recommended to check the table of sterilization cycles below, page 9.;

1. Connect the plug into the 230V outlet. Switch on the master (A).
The display shows a short self test, including the Firmware Version.
The yellow pilot lamp (B) is turned on in order to show that the machine is ready for the cycle.
2. Place the material to be treated inside the sterilizer and close the door (do not open the door during the cycle).
3. Set the desired temperature and time of sterilization,
 - **Set the temperature** : with the temperature pilot lamp on, press the **+** or **—** keys, or keeping the push button pressed, the increase or the decrease of the value becomes quick. After reaching the desired value, wait 3 seconds for the storing of the set value.
 - **Set the sterilization time**: press the switch push button  the timer pilot lamp turns on  . Repeat the previous operations, in order to visualize and modify the time. Wait 3 seconds for the storing of the set value.

4. After the parameters setting, start the sterilization cycle by pressing the "START" push button. The yellow pilot lamp (B) turns off.
5. After some minute the sterilizer arrives at the set temperature and keeps it for the set time.
6. The temperature measured inside the sterilization chamber is visualized on the display (with a + 10°C tolerance) or, by switching the ⇄ push button, the remaining time to the end of the cycle.
7. At the end of the set time, the cycle automatically stops and the display shows the writing END matched to few BIP sounds, informing that the sterilizer has finished the cycle.

IMPORTANT: The cycle is considered valid only if, at the end, the writing END is appearing.

8. If the equipment is not used, turn the master switch (A) off.

ATTENTION: AT INFINITE TIME THE EQUIPMENT CAN BE SWITCHED OFF ONLY MANUALLY.

STERILIZATION CYCLES

Temperature	Time in minutes
180°C	*45'
170°C	*75' (1h 15')
160°C	*120'

* minimum recommended time

Cleaning

To carry out a good cleaning use only neutral detergents and rinse with plenty of water. The gaskets and all plastic parts can be cleaned with a cotton flock soaked in alcohol.

Do not use sponges or abrasive materials which can damage the surface.

To clean do not use detergents, disinfectants or other liquids containing chlorides as they are corrosive for stainless steel; in case of contact please rinse immediately.


ALARM TABLE

Display/Defect	Cause	Remedy
AL 1 – Alarm heating element broken	1) Excessive load 2) Heating element broken	1) Reduce the load and repeat the cycle 2) Replace the faulty heating element
AL 2 – Alarm Low Temperature	Heating element broken	Replace the faulty heating element
AL 3 – Alarm Overheating	CPU card in Error	1) Open immediately the door 2) Replace the faulty CPU card

AL 4 - Probe broken	CPU card in Error	Replace the faulty CPU card
AL 5 – Black Out	The Black Out	Repeat the cycle
The equipment is completely stopped	1)Over-temperature, the safety thermostat is working 2) The fuses is broken 3) CPU card in Error	Open immediately the door 1) Reset the safety thermostat 2) Replace the fuses 3) Replace the faulty CPU card

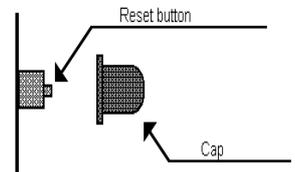SAFETY: Should the safety thermostat intervene, all power goes off except to the green main switch. In order to reset the sterilizer it is necessary to unscrew the black cap situated on the back side of the machine and push the button its full course. The overheating cut-off is operating because of an excess temperature in the chamber. If during the following cycle the overheating cut-off should work again, call the technical assistance in order to eliminate the inconvenience causing its intervention.

WARRANTY

This instrument is designed and built according to the Community Directive CEE 93/42 “Medical equipments” and C.B.M. is responsible for safety, reliability and performances provided that possible modifications or repairs are carried out by authorised people, the corresponding electric plant complies with IEC regulations and the equipment is used in compliance with these instruction for use.

C.B.M. guarantees its sterilizer for two (2) years, in case of manufacturing defects or failures of components, not due, according to C.B.M. evaluation, to the operators’ wrongful use of the apparatus.

The warranty does not cover parts subject to wear and repairs or replacements carried out by personnel not authorised by C.B.M.

Details, descriptions and pictures are merely indicative.

C.B.M. S.r.l. reserves the right to carry out ameliorative modifications to its own products without any previous notice.

Disposal Of Electric And Electronic Refuse

Waste from Electrical and Electronic Equipment - Directive 2012/19/UE (WEEE)

The symbol (shown above) that appears on this product means that it should not be disposed of together with normal household refuse but should be treated separately.

Electric and electronic equipment may contain substances that are harmful to the environment and to the health of humans and should therefore be disposed of at special waste disposal sites.

There are harsh legal penalties in force against persons found disposing of such equipment in an irresponsible manner.

When the equipment is no longer of any use, it may taken back to the outlet from which you intend to purchase a replacement.

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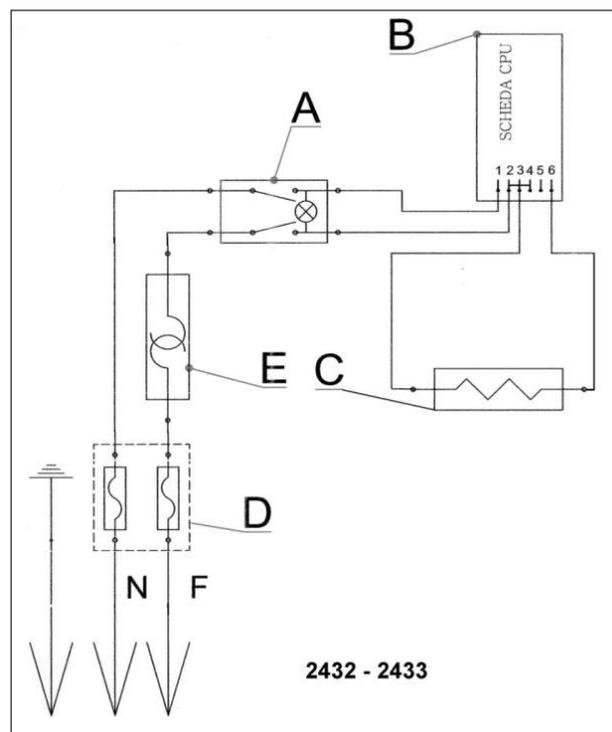
Safety table

Apparatus	Dry sterilizer
Manufacturer	C.B.M. S.r.l. via Castello n°10 26038 Torre de Picenardi Cremona - Italy
Model	2432, 2433
Reference norm	CEI EN 61010-1
Data of plate:	
Attached documentation	yes
Terminals	
Protection ground wire	yes
equipotential	no
Functional ground wire	no
Feeding plug	
Nominal current	10A
Disassembled type	no
Security mark	yes
Feeding cable	separable
Fuses	
Open to outside	yes
Type	F
Characteristics	
Class equipment	I
Installation category	II
Pollution degree	1
Security mark	no
Self-certification	yes
Max. operating altitude	2000 m.
Operative environment temperature	5°C up to 40°C
Max relative humidity in the operative environment	Max 93 %
Storage conditions	0°C ~ + 50°C

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We: C.B.M. S.r.l., Via Castello 10, 26038 Torre de Picenardi (CR) Italy, in the person of its Administrator and Legal Representative Mr. Carlo Busatti declare on our own responsibility, that the following electrical equipment:

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meet the provisions of following Directives which apply to them and its modifications:

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